

# **BloodSTOP® iX Advanced Hemostat with WoundHEAL®**

## *For ENT Otolaryngology*

“BloodSTOP® iX was simple to use, less expensive than other hemostatic products and was well tolerated by the patients.”

- M. Shapiro, D.O., FOCOO-HNS  
ENT/Facial Plastic Surgery

**Epistaxis**

**Tonsillectomy and/  
or Adenoidectomy**

**Septoplasty**


**Functional Endoscopic Sinus  
Surgery (FESS)**

**Functional Rhinoplasty**


***Power to control bleeding  
Power to heal***

BloodSTOP iX has European Union CE Certification for implant surgical and topical use, including when used with anticoagulant medications. BloodSTOP iX currently has USFDA clearance for topical wound hemostasis.

**ADVANCED HEMOSTAT  
WITH WOUNDHEAL®**

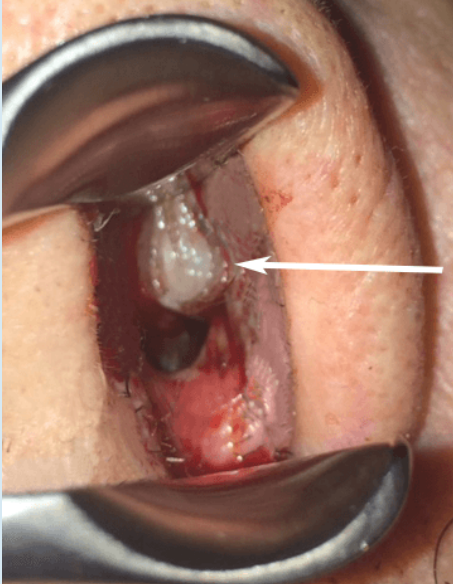

Multilayer BloodSTOP® iX for control of anterior nosebleed

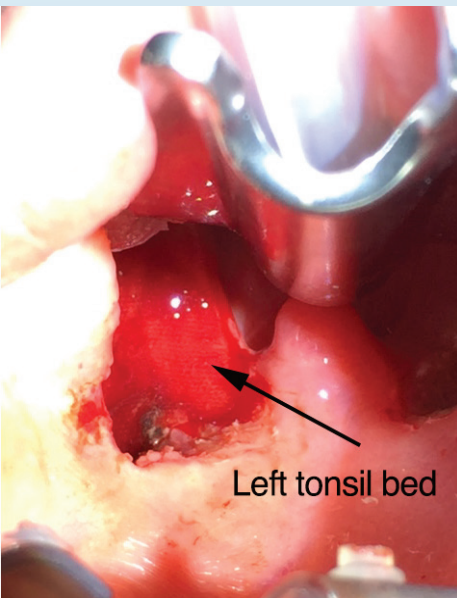

BloodSTOP® iX in Tonsillectomy

BloodSTOP® iX Advanced Hemostat with WoundHEAL® is a clinically proven revolutionary hemostatic dressing and wound healing matrix that provides rapid hemorrhage control, reducing blood loss. Unlike other wound care products with the single benefit of hemostasis, BloodSTOP iX also seals the wound to prevent contamination, providing a moist wound environment for proliferation of new tissue, making BloodSTOP iX a total wound care solution. It is an all-natural, plant-based, biocompatible, water soluble etherified sodium carboxymethyl cellulose matrix that utilizes patented technology to achieve superior hemostasis. It contains no animal, human, or added-chemical components.

At the wound interface, upon contact with blood, exudates, or body fluids, BloodSTOP iX quickly turns into a translucent sticky gel, and activates the intrinsic coagulation cascade, thus accelerating clot formation. The protective gel seals the wound without irritation, and minimizes blood loss.

BloodSTOP iX is reliable and fast acting which allows the ENT surgeon to focus on the operative tasks with confidence in knowing that the bleeding is under control.

**FEATURES & BENEFITS**

- Rapid hemostasis
- Translucent gel for easy wound monitoring
- The gel sticks, seals & protects wounds
- Prevents contamination, conforms to and adheres to wound sites
- pH neutral: ~7.2, biocompatible
- Easily applied and removed without irritation
- Provides a moist wound environment to promote healing
- Water solubility allows for easy removal of dirt and foreign bodies with irrigation
- Leaves no residue behind
- Highly cost effective

Product Description	Size	Quantity	Product Code
BloodSTOP® iX Advanced Hemostat with WoundHEAL®	2" × 2" (5 × 5cm)	12 pc/box	BS-iX-14
	2" × 4" (5 × 10cm)	12 pc/box	BS-iX-15

**LifeSciencePLUS, Inc.**

2520A Wyandotte Street, Mountain View, CA 94043, USA

Mailing: P.O. Box 60783, Palo Alto, CA 94306

1.877.587.5433 (U.S.) 1.650.565.8172 (International)

www.lifescienceplus.com; www.bloodstopix.com

© 2021 LifeScience PLUS, Inc. All Rights Reserved.

BloodSTOP® is a registered trademark. BloodSTOP® and BloodSTOP® iX are covered by U.S.

Patent No. 7,262,181