

DonorSeal™

Powered by BloodSTOP® iX

Advanced Hemostatic and Wound Healing Dressing for Donor Sites

100% natural plant-based

Instructions for Medical Professionals

Indications for Use

• Emergency and therapeutic use in the control of bleeding from the skin and other surface wounds where temporary control of bleeding is required.

Directions

1. Apply DonorSeal Matrix to the donor site; it will turn into a gel upon contact with blood and exudate to seal the wound. Use additional layers of DonorSeal as needed.
2. Place a non-adherent secondary dressing over DonorSeal to keep moist.
3. Wrap sterile gauze over the non-adherent dressing, followed by an ace bandage for compression. Change the outer gauze daily or as needed for drainage.
4. Remove all dressings, gently clean the site with sterile water or sterile saline, being careful not to disrupt the wound healing.

WARNINGS:

- Longer compression times may be required for patients who are obese, hypertensive, or are on anticoagulants.
- When removing DonorSeal and/or any secondary dressing, be careful not to disturb the clot.
- Never peel DonorSeal from a wound, as re-bleeding could occur. If DonorSeal dries out, soak thoroughly to rehydrate before removal.
- In case of swelling or redness, discontinue use.
- Do not reuse, as device will no longer be sterile after initial use, and could result in infection.
- Cover DonorSeal with a wrap or adhesive bandage to maintain pressure if the patient is to be moved.

CONTRAINDICATIONS:

- Not for use on chemical burns resulting from strong acids or bases.
- Do not use on patient with known cellulose allergy.

STERILE: Sterility guaranteed unless individual wrapper is opened or damaged.

STORAGE: For maximum shelf life, storage temperature of 15-30 °C (59-86 °F) is recommended.



STERILE R



Not made with natural rubber latex

Symbols Used on Labeling



Single use



Company address



Use by date



Do not use if package is damaged



Reference number



Method of sterilization: irradiation



Batch number



Do not re-sterilize



See instructions for use



Temperature Limits



Manufacturer:

LifeScience PLUS, Inc.
P.O. Box 60783, Palo Alto, CA 94306
2520A Wyandotte St., Mountain View, CA 94043
Toll Free: 1-877-587-5433 - U.S.A.
1-650-565-8172 - International
Fax: 1-650-336-1130
www.LifeSciencePlus.com
Email: info@lifescienceplus.com
Manufactured in the USA

LifeScience PLUS
REVOLUTIONARY WOUND CARE

© 2024 LifeScience PLUS, Inc.
All Rights Reserved.

IFU-DS-US
Rev. 2024-07-25